



KLS Martin ME 411

Utmost Safety and Reliability

Thanks to an Advanced Safety Concept

10 different types of current for a broad range of indications



Monopolar cutting 1
Pure cutting current allowing smooth cuts, with optional integration of a starting cut pulse.



Monopolar cutting 2
Cutting current blended with coagulation current.



Cutting URO 1
Cutting current allowing smooth cuts in liquid media (e.g. TUR), with optional integration of a starting cut pulse.




Cutting URO 2
Cutting current blended with coagulation current, for use in liquid media.



Contact coagulation 1
Coagulation current with very deep-reaching effects; electrode in direct contact with the tissue.




Contact coagulation 2
Coagulation current with deep-reaching effects; electrode in direct contact with the tissue. Particularly suitable in TUR.

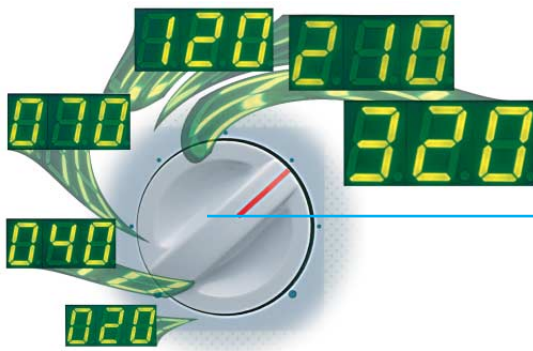

Progressively increasing/ decreasing power output

In the lower output range, the user can select the desired HF output power value in 1-watt steps, which allows very precise dosing. As the power adjuster is rotated clockwise, output power increases on a non-linear, progressive basis. This means that high power reserves are available in the upper power range. Thanks to these unique power-range and performance characteristics, the Martin ME 411 is universally applicable.

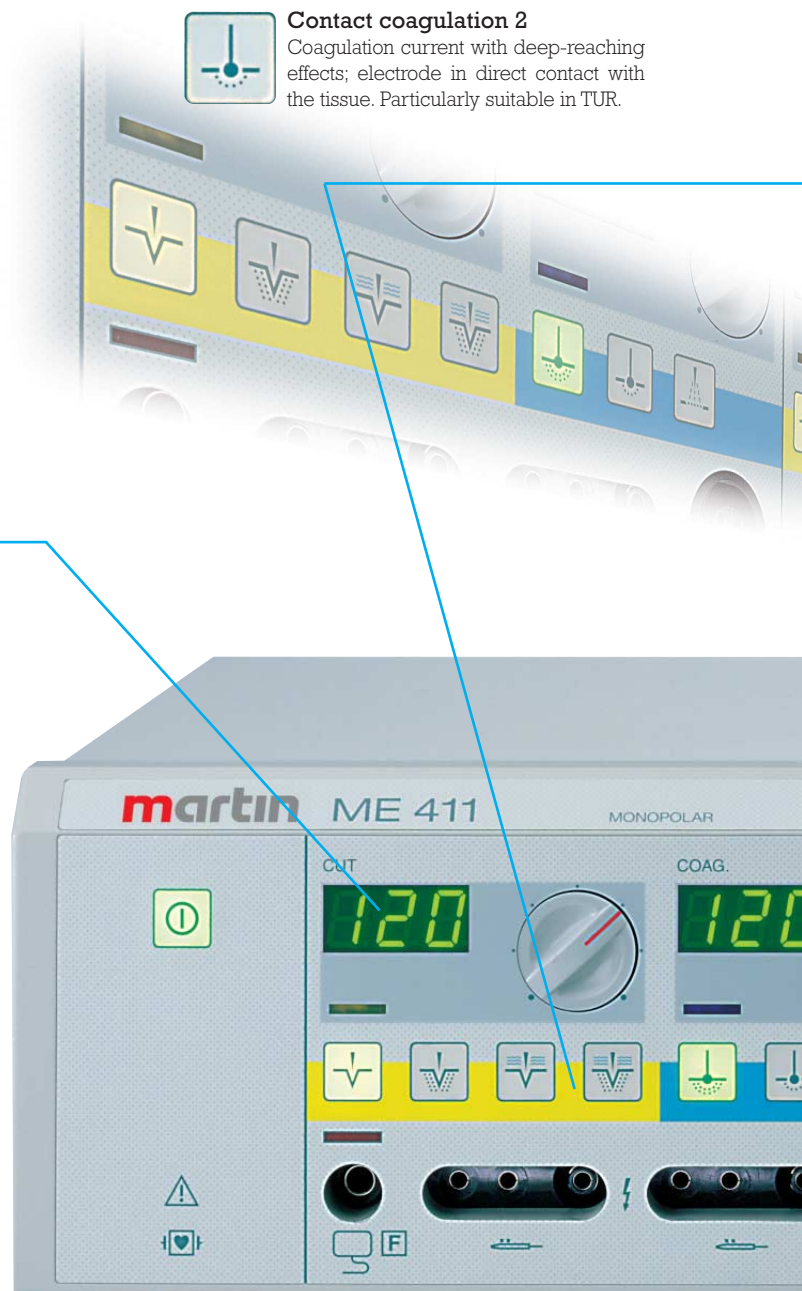

Advanced safety concept based on two microprocessors working in parallel



Spray coagulation

Non-contact surface coagulation (fulguration). In tissue contact under liquid media (e.g. TUR) useful for staunching micro-bleedings.



Bipolar cutting 1

Pure bipolar cutting current, to be used with sling-type electrodes, for example.



Bipolar cutting 2


Bipolar cutting current blended with coagulation current to be used f. e. with electro-surgical forceps, for example.



Bipolar coagulation


Automatic bipolar coagulation to be used with bipolar tweezers or bipolar instruments.

The tasks of monitoring and controlling the various system components are shared by two microprocessors working in tandem. As these two modules also monitor each other, they provide the basis for Martin's highly advanced safety concept.

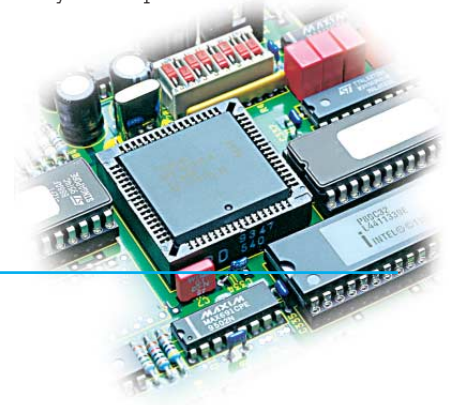

Multi-function connector for connecting intelligent accessories

The bipolar output port of the Martin ME 411 has been designed as a multi-functional interface for connecting intelligent accessory equipment. Thus, this interface provides for proper information transfer between the HF generator and the connected accessory part. Being upward compatible, this interface also takes account of future developments.

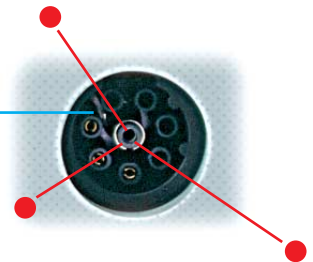

Integrated dynamic, self-adaptive HF power output control

With the help of the dynamic, self-adjusting power output control that has been integrated into the system, the Martin ME 411 automatically selects the most suitable operating point. This guarantees optimum cutting results irrespective of tissue impedance or cutting speeds.


Highest patient safety thanks to Patient Control System (PCS)

Using a dual-pad neutral electrode, the integrated Martin Patient Control System (PCS) continually monitors the neutral electrode for proper application. So if the electrode is only insufficiently in contact with the patient's skin, PCS inhibits HF-power and gives alarm, before critical situations arise. Moreover, the system also monitors single-pad neutral electrodes in acc. with IEC 601-2-2.




Better protection against mal-operation by PIN Error Control

The Martin ME 411 performs a power-on-self-test routine whenever the system has been switched on. If an error or malfunction is found, HF power output is blocked and the respective error code is displayed.

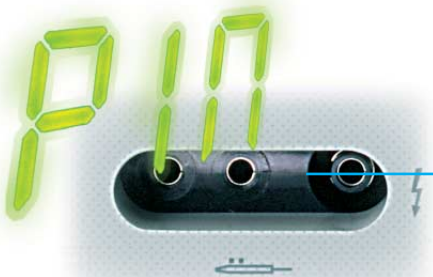

Highest safety for the patient by patented active leakage current compensation

The Martin ME 411 monitors and actively controls the HF leakage currents and compensates them. This helps to avoid the risks associated with so-called „accidental HF burns“, thus providing additional safety to the patient.

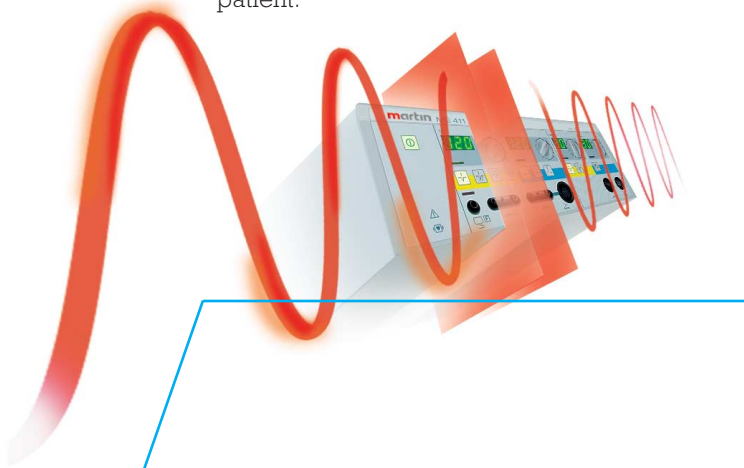

Bipolar Coagulation with AUTO-START and AUTO-STOP Function


The bipolar coagulation current is activated automatically by tissue contact and will automatically cut off the bipolar HF current as soon as the coagulation process has been completed, prior to unwanted side-effects like tissue glueing or pollution.



Electrosurgical Unit ME 411

Utmost Safety and Reliability

Thanks to an Advanced Safety Concept

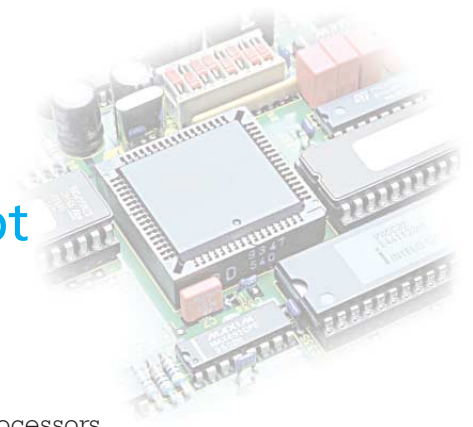

Advantages of the Martin ME 411

- Utmost patient safety by constant monitoring and active compensation of HF-leakage currents
- Constant monitoring of neutral electrode for proper application via PCS
- Safety of operation due to power-on-self-test, including detailed error messages
- Digital setpoint value display in accordance with the selected output power level
- Integrated dynamic, self-adaptive HF output control for reproducible cuts
- Progressive power output adjustment
- 10 different types of current available for monopolar and bipolar application
- Two independent power generators
- Two parallel μ -processors
- Two URO-CUT Functions particularly for underwater-cutting
- Serial interface for MABS
- Comfortable bipolar Cut or Coagulation
- Bipolar Autostart/Autostop function
- Symmetrical Bipolar Output, significantly reducing tissue glueing or pollution of electrodes
- Bipolar instrument recognition for use of intelligent accessories
- Broad range of high-quality, practice-oriented accessories – see Martin Accessories catalog

Martin Argon-Beamer-System (MABS)

In combination with the Martin Argon Beamer MB 181 the Martin HF-unit ME 411 with its intelligent interactive interface forms the Martin Argon-Beamer system. The MABS inaugurates new applications for superficial soft coagulation of tissue in open or endoscopic surgery.



Martin ME 411 Specifications

Supply voltage	100 V / 115 V / 127 V / 230 V / 240 V, 50 Hz - 60 Hz Adjustable by the Technical Service		
Power input	in standby mode: 1 VA with no HF power output: 42 VA at max. power output: 880 VA		
Class of protection	I		
Classified acc. to MDD	IIb		
Leakage currents	in acc. with IEC 601, Part 2-2		
Type of equipment	CF; defibrillator-proof		
Nominal frequency	400 kHz		
Pulse frequencies	30 kHz at cut 2, URO cut 2 and contact coagulation 1 65 kHz at contact coagulation 2 50 kHz at spray coagulation 30 kHz at bipolar cutting 2		
HF Power		CF	
Cutting 1	max. 320 W at 350 ohms	1.8	max. 2700 Vss
Cutting 2a	max. 320 W at 350 ohms	2.3	max. 3500 Vss
Cutting 2b	max. 320 W at 800 ohms	2.6	max. 4300 Vss
Urolog. cutting 1	max. 320 W at 350 ohms	1.8	max. 2700 Vss
Urolog. cutting 2	max. 320 W at 800 ohms	2.6	max. 4800 Vss
Contact coagulation 1a	max. 250 W at 200 ohms	1.8	max. 1300 Vss
Contact coagulation 1b	max. 250 W at 200 ohms	3.0	max. 2600 Vss
Contact coagulation 2	max. 150 W at 500 ohms	5.4	max. 4300 Vss
Spray coagulation	max. 120 W at 1000 ohms	5.5	max. 6000 Vss
Bipol. cutting 1	max. 80 W at 500 ohms	1.8	max. 900 Vss
Bipol. cutting 2	max. 80 W at 500 ohms	2.1	max. 1200 Vss
Bipol. coagulation	max. 80 W at 100 ohms	1.8	max. 400 Vss
Duty type	intermittent (10s/30s = duty factor of 25%)		
Sound level	HF activation 55 dB(A), (adjustable 50 dB - 60 dB by Technical Service) Alarm: 65 dB(A)		
Weight	14.2 kg		
Interference suppression	limits in conformity with EN 55011 interference immunity in conformity with IEC 801 CE marking conform with 93/42/EEC		
Dimensions	405 mm x 135 mm x 380 mm (WxHxD)		

Ordering Data

80-041-01-04	Martin electrosurgical unit ME 411, with main cable, no accessories included
80-140-00-04	Manual-switch accessories set, large
80-140-01-04	Foot-switch accessories set, large
80-140-02-04	Manual-switch accessories set, small
80-150-00-04	Accessory Set bipolar

KLS Martin Group

KLS Martin France SARL
68200 Mulhouse · France
Tel. +33 3 89 51 3150
france@klsmartin.com

KLS Martin UK Ltd.
Reading RG1 3EU · United Kingdom
Tel. +44 1189 000 570
uk@klsmartin.com

KLS Martin do Brasil Ltda.
CEP 04.531-011 São Paulo · Brazil
Tel.: +55 11 3554 2299
brazil@klsmartin.com

Gebrüder Martin GmbH & Co. KG
Representative Office
121471 Moscow · Russia
Tel. +7 499 792-76-19
russia@klsmartin.com

Martin Italia S.r.l.
20864 Agrate Brianza (MB) · Italy
Tel. +39 039 605 6731
italia@klsmartin.com

Nippon Martin K.K.
Osaka 541-0046 · Japan
Tel. +81 6 6228 9075
nippon@klsmartin.com

KLS Martin Australia Pty Limited
Artarmon NSW 2064 · Australia
Tel.: +61 2 9439 5316
australia@klsmartin.com

Gebrüder Martin GmbH & Co. KG
Representative Office
201203 Shanghai · China
Tel. +86 21 5820 6251
china@klsmartin.com

Martin Nederland/Marned B.V.
1271 AG Huizen · The Netherlands
Tel. +31 35 523 45 38
nederland@klsmartin.com

KLS Martin L.P.
Jacksonville, FL 32246 · USA
Tel. +1 904 641 77 46
usa@klsmartin.com

KLS Martin SE Asia Sdn. Bhd.
14100 Simpang Ampat, Penang · Malaysia
Tel.: +604 505 7838
malaysia@klsmartin.com

Gebrüder Martin GmbH & Co. KG
Representative Office
Dubai · United Arab Emirates
Tel. +971 4 454 16 55
middleeast@klsmartin.com

Gebrüder Martin GmbH & Co. KG
A company of the KLS Martin Group
KLS Martin Platz 1 · 78532 Tuttlingen · Germany
Postfach 60 · 78501 Tuttlingen · Germany
Tel. +49 7461 706-0 · Fax +49 7461 706-193
info@klsmartin.com · www.klsmartin.com

